

# Short instructions for disinfection of measuring prisms

English



Excerpt from HAAG-STREIT instruction for use 'cleaning and disinfection'. Complete instructions for use: [www.haag-streit.com](http://www.haag-streit.com)




**FORBIDDEN!** Never use damaged measuring prisms. **NOTE:** If using 'Tristel Pre-Clean Wipes', 'Tristel DUO' and 'Tristel Rinse Wipes', refer to the manufacturers instructions for use.


## Cleaning




1

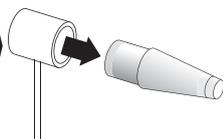


2

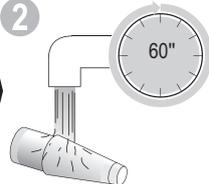


3

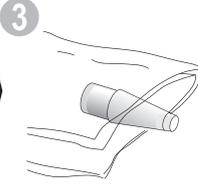

4

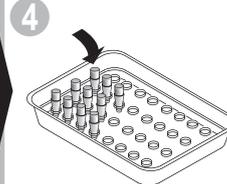

5

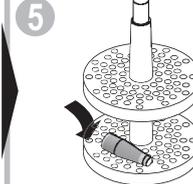


- Disposable gloves recommended (personal security).
1. Remove measuring prism from the holder.
  2. Clean for 60 seconds under running water. If necessary, use a pH-neutral cleansing agent.

- Rinse off the cleansing agent under running, cold drinking water for 60 seconds.
- Repeat this cleaning process if visible contamination is still present.
3. Dry with a lint-free disposable cloth.
  - Perform a visual inspection for cleanliness and integrity.

4. Place the cleaned measuring prisms in the Desinset plastic tray.
  5. Remove the measuring prisms from the plastic tray and place them on the lower level of the Desinset plastic insert.
- Measuring prisms must not overlap.

## Disinfection / rinsing


6

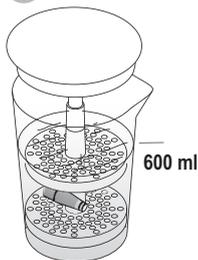


7



8

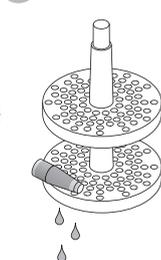


9

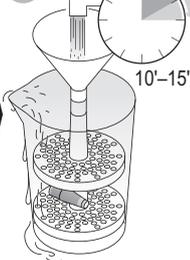

## Drying / storing

10

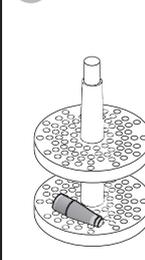

11



12



6. Add at least 600 ml of disinfectant solution.
- Measuring prisms must be completely immersed in the disinfectant solution.
7. Maximum material-compatible exposure time: 60 minutes.

8. Allow the plastic insert to drip dry and place it in the other measuring beaker.
9. Place the funnel on the plastic insert.
- Rinse the contents under running water for a minimum of 10 minutes and a maximum of 15 minutes.

10. Remove the plastic insert with the measuring prisms.
11. Dry each measuring prism individually using a soft disposable cloth.
12. Store the measuring prisms in a closed Desinset glass container.



### NOTE!

The disinfectant must be replaced as follows:

- When visibly soiled
- Once a day or as specified by the manufacturer
- List of Disinfectants: [www.haag-streit.com](http://www.haag-streit.com)
- HAAG-STREIT recommends Desinset



### FORBIDDEN!

- Disinfection with alcohol
- Cleaning with acetone
- Disinfection using UV radiation
- Sterilization using steam or ethylene oxide
- Temperatures above 60°C

**HAAG-STREIT**  
**DIAGNOSTICS**

Haag-Streit AG  
Gartenstadtstrasse 10  
3098 Koeniz, Switzerland  
[www.haag-streit.com](http://www.haag-streit.com)